



OWNER'S MANUAL

MEDICAL MONITOR

Please read this manual carefully before operating your set and retain it for future reference.

32HS710S



www.lg.com

Copyright © 2025 LG Electronics Inc. All Rights Reserved.

CONTENTS

ON CLEANING	3
LICENSE	4
OPEN SOURCE SOFTWARE NOTICE INFORMATION	4
ASSEMBLY AND PREPARING	5
USER SETTINGS	15
TROUBLESHOOTING	28
PRODUCT SPECIFICATIONS	30
EXTERNAL CONTROLLER SETUP	36
PREVENTIVE MAINTENANCE PLAN – RECOMMENDATION	49
CYBER SECURITY MANAGEMENT SERVICE.....	50

Be sure to see the regulatory manual provided for more information about the proper use and safety of the equipment.

Please note that this information is for proper use and safety of the equipment. The following symbols may indicate a hazardous situation in which, if not heeded, may result in serious injury or even death to the user or others, or damage to the equipment.

WARNING

- Indicates warning and safety instructions. If not adhered to, it could result in death or serious injury to the user or others.

CAUTION

- Indicates a hazardous situation which, if not heeded, may result in minor or moderate injury to the user or others, or damage to the equipment.

ON CLEANING

Recommended Cleaning Chemicals

- Isopropanol 70 %
- Ethanol 70 %
- 0.9 % NaCl solution
- Biospot 500 ppm

How to Use Cleaner

- Prior to cleaning, turn off the monitor and remove the power cable.
- Slightly dampen a soft non-abrasive cloth with a recommended cleaner, then lightly rub the screen with no more than 1 N of force.
- The cleaner could cause serious damage if it leaks inside the monitor while cleaning.
- Do not use benzene, thinner, acids or alkaline cleaners or other such solvents.
- The screen should only be cleaned by authorized personnel, using the above guidelines.
- Visually inspect the product for any contamination after cleaning. If contamination remains, repeat the cleaning process.
- If discolouration/cracks appear after cleaning, discontinue use and contact the manufacturer.
- Ensure lighting is adequate during the cleaning process.

LICENSE

Each model has different licenses. Visit www.lg.com for more information on the license.



The terms HDMI, HDMI High-Definition Multimedia Interface, HDMI trade dress and the HDMI Logos are trademarks or registered trademarks of HDMI Licensing Administrator, Inc.



The SuperSpeed USB Trident logo is a registered trademark of USB Implementers Forum, Inc.

"DICOM" is the registered trademark of the National Electrical Manufacturers Association for its Standards publications relating to digital communications of medical information."

OPEN SOURCE SOFTWARE NOTICE INFORMATION

To obtain the source code that is contained in this product, under GPL, LGPL, MPL, and other open source licenses that have the obligation to disclose source code, and to access all referred license terms, copyright notices and other relevant documents, please visit <https://opensource.lge.com>.

LG Electronics will also provide open source code to you on CD-ROM for a charge covering the cost of performing such distribution (such as the cost of media, shipping, and handling) upon email request to opensource@lge.com.


This offer is valid to anyone in receipt of this information for a period of three years after our last shipment of this product.

ASSEMBLY AND PREPARING


Product Composition

Please check whether all the components are included in the box before using the product. If there are any missing components, contact the retailer where you purchased the product.


Note that the product and related components may look different from those shown here.

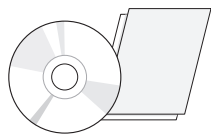


Owner's Manual/Regulatory Manual/Cards

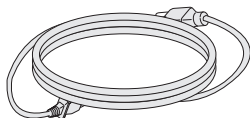

Power Cord

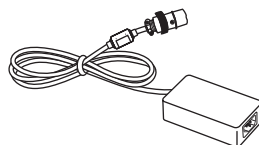

AC/DC Adaptor

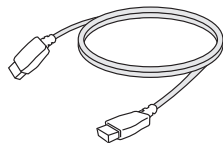

HDMI Cable



DisplayPort Cable



Screw (M4 x L12)



Screw cap x 8

CAUTION

- Always use genuine LG components to ensure safety and product performance.
- The product warranty will not cover damage or injury caused by the use of unauthorized components.
- It is recommended to use the supplied components.
- If you use generic cables not certified by LG, the screen may not display or there may be image noises.
- Only use components listed below that meet the listed standard. Unauthorized components may cause damage to and malfunction of the product.

Component	Standard
HDMI Cable	UL, Impedance 100 ohm
DisplayPort Cable	UL, Impedance 100 ohm
Power Cord	US – Approved Medical grade regulation Others – Approved country safety regulation

- Except for the components listed above, only use AC/DC adaptors and components supplied by the manufacturer.

NOTE

- The components may look different from those illustrated here.
- Without prior notice, all product information and specifications contained in this manual are subject to change to improve the performance of the product.
- To purchase optional accessories, visit an electronics store or an online shopping site, or contact the retailer from which you purchased the product.
- The power cord provided may differ depending upon the region.



Supported Software

Check the drivers and software supported by your product and refer to the manuals on the CD enclosed in the product package.

Drivers and Software	Installation Priority
LG Calibration Studio	Recommended

- Required and Recommended: You can download and install the latest version from the enclosed CD or from the LGE website (www.lg.com).

Product and LED Control buttons

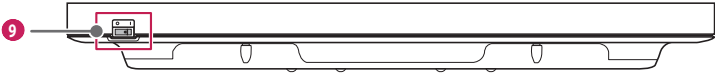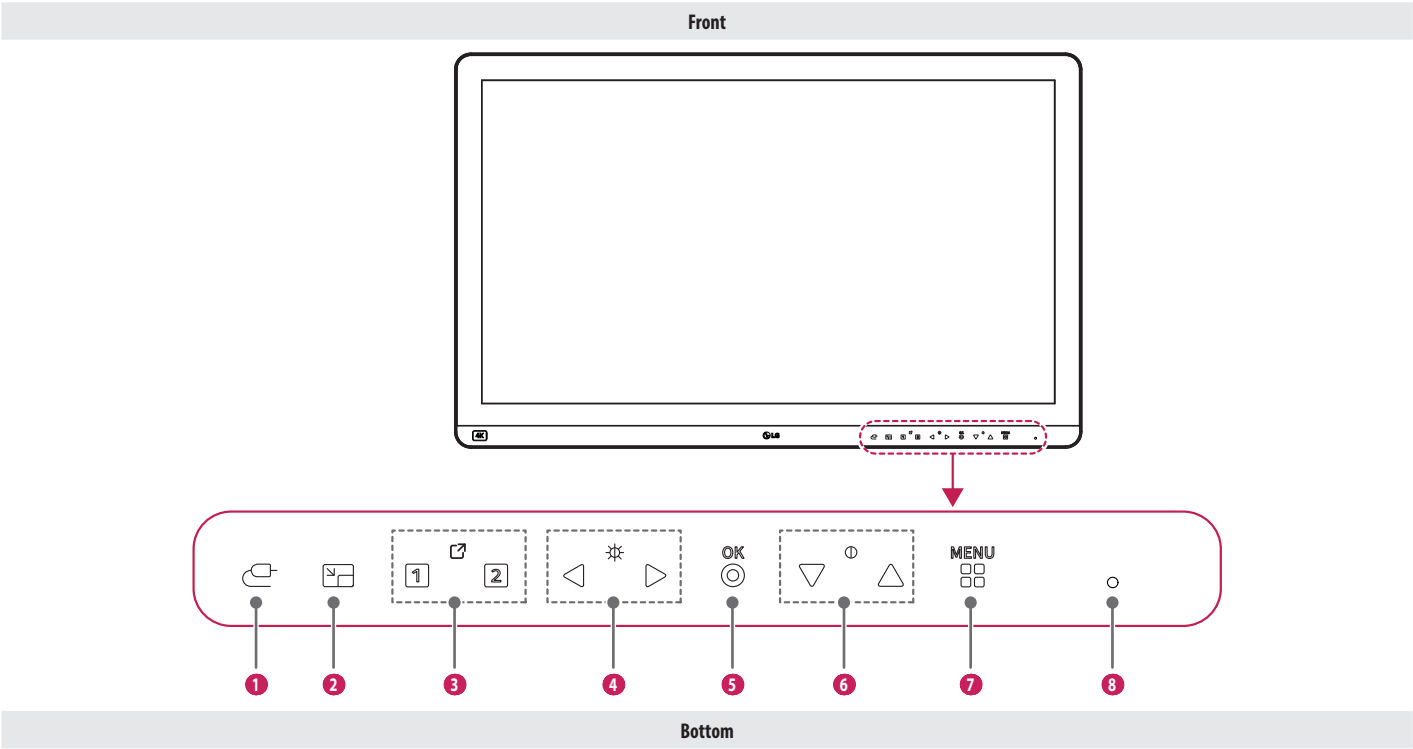

LED Control button Functions


1	Input • Switch input mode.
2	PIP/PBP • Adjusts display mode for 2 or more screens.
3	Hotkey • Switch to the item set in [Hot key Settings].
4	Brightness • Adjusts the screen brightness.
5	OK • Selects and confirms menus or options
6	Contrast • Adjusts the screen contrast.
7	Menu • Displays LED Control buttons on the front panel and opens the menu mode.
8	Power indicator • The green indicator illuminates if the power is on.
9	Power switch • Turns the power on/off.

! NOTE

- The Power button is located at the bottom left on the front of the monitor.
- If the Control Key LED is turned off, press the Control Key (⏏) button to turn the Control button LED on. When the Control Key LED is turned on, you can control the Control Key functions.

Connectors

Back

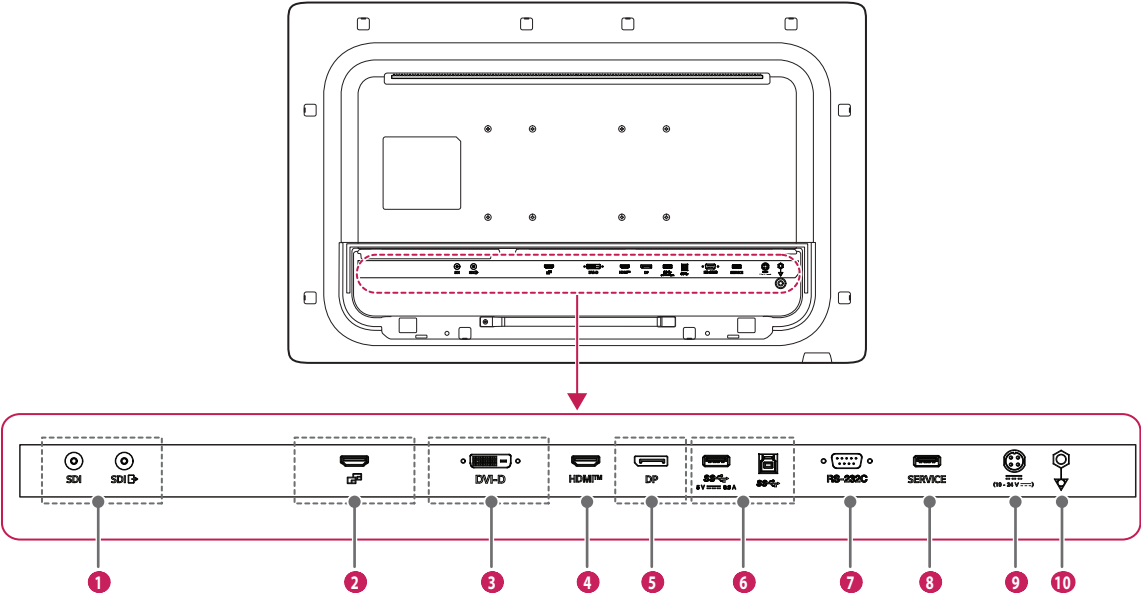

1	SDI IN/SDI OUT Receives or transmits digital video signals.
2	CLONE SCREEN - Clones the screen currently being output and outputs it with the HDMI signal. - The screen is output only when the monitor is turned on. - The signal will not be output if the monitor's input signal contains HDCP Contents. - The resolution, refresh rate, and color gamut of the output screen can be adjusted. - See page 19 of the user manual for more information. - The output signal does not support audio playback.
3	DVI-D IN Input digital video signal.
4	HDMI IN - Input digital video signal. - Using a DVI to HDMI / DisplayPort to HDMI cable may cause compatibility issues. - Use a certified cable that displays the HDMI logo. The screen may not appear or a connection error could occur if a non-certified cable is used. - Recommended HDMI cable types - High speed HDMI[®]/TM cable - High speed HDMI[®]/TM Ethernet cable
5	DP IN - Receives digital video signals. - A cable with DisplayPort 1.2 specifications is recommended when using a Mini DisplayPort to DisplayPort cable.

6	USB IN/USB UP - Used for HW Calibrator (Optional Accessory) connection. - Connect peripheral device to the USB input port. - A keyboard, mouse, or USB storage device can be connected. - To use USB 3.0, connect the A-B type USB 3.0 cable to the PC.  CAUTION Cautions when using a USB storage device. - A USB storage device which has a built-in automatic recognition program or uses its own driver might not be recognisable. - Some USB storage devices may not be supported or may not work properly. - It is recommended to use a USB hub or hard disk drive with power supplied. (If the power supplied is not enough, the USB device may not be detected properly.)
7	RS-232C Connect the RS-232C terminal with an external device to control the monitor.
8	SERVICE ONLY This USB port is used only for service.

9	DC-IN • Connect an AC/DC adapter. • The output of the enclosed adaptor is 19 V. • The product is designed to be used with adaptors that have an output ranging between 19-24 V. Please use the adaptors of medical standards.
10	Potential Equalization Conductor • Connect an equipotential plug.

! NOTE


- All signal out terminals (SDI, etc.) output a signal when the power switch of the monitor is on. When the power switch is off, no signal is outputted.
 - This monitor supports the *Plug and Play feature.
- * Plug and Play: A feature that allows you to add a device to your computer without the need for physical device configuration or user intervention.
- The specifications of the SDI output terminals for transmitting a screen
 - SDI OUT: Connect a 100-meter cable (BELDEN 1694) to transmit a duplicated screen to a monitor.
 - CLONE SCREEN: The clone screen can be transmitted to one monitor by connecting the 1.8-metre cable provided.

Installing the Monitor


Cable connection and organize

Before connecting the connectors, remove the back door as shown below.


- 1 Lift the back door in the direction of the arrow below to remove it. The back door is held in place with a magnet.

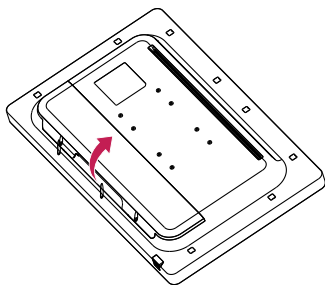

- 2 Remove the cable holder screws tightened on the back of the monitor. After installing the cables, tidy them up using the cable holders. Once the cables are tidy up, assemble it using the cable holder screws.

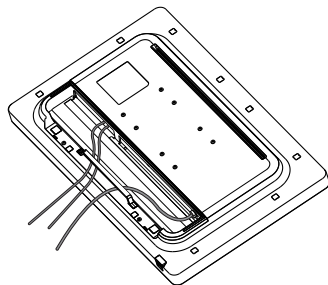

- 3 Once the back door is attached, assemble it using the screws.

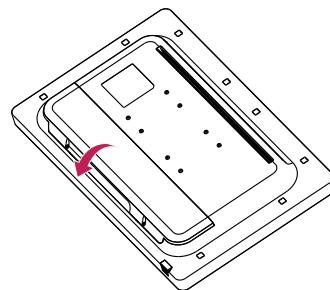


⚠ CAUTION


- When the back door is attached to the monitor, the monitor meets water resistance standards. Do not use the monitor with the back door detached, as the water resistance capability is not guaranteed without the door.
- Only fix the back door with screws when the potential equalization conductor is not connected. When the potential equalization conductor is connected, don't fix the back door with screws (you can cover the back door with a magnet only).

Installing on the Wall

Install the monitor at least 100 mm away from the wall on each side of the monitor to ensure sufficient ventilation. Detailed installation instructions can be obtained from your local dealer. Please refer to the manual to install and set up a tilting wall mounting bracket.

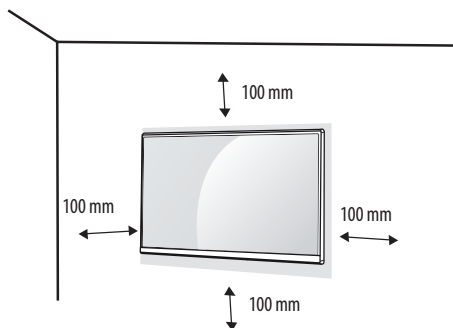


To install the monitor to a wall, attach a wall mount plate (optional) to the back of the monitor. Make sure that the wall mount plate (optional) is securely fixed to the monitor and to the wall.

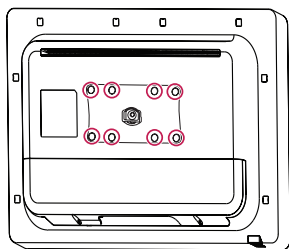

Secure the wall mount screw and attach the screw cap.

Installing the Wall Mount Plate

This monitor meets the specifications for the wall mount plate or other compatible devices.

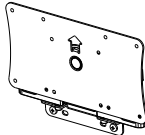
Wall Mount (mm)	100 x 100
Standard screw	M4 x L10
Required screws	4
Wall mount plate (optional)	LSW140 

! NOTE

- A non-VESA standard screw may damage the product and cause the monitor to fall. LG Electronics is not liable for any accidents relating to the use of non-standard screws.
- The wall mount kit includes the installation guide and all necessary parts.
- The wall mounting bracket is optional. You can obtain optional accessories from your local dealer.
- The length of the screw may differ for each wall mounting bracket. If you use screws longer than the standard length, it may damage the inside of the product.
- For more information, please refer to the user manual for the wall mounting bracket.
- Be careful not to apply too much force while mounting the wall mount plate as it may cause damage to the screen.
- Remove the stand before installing the monitor on a wall mount by performing the stand attachment in reverse.

⚠ CAUTION

- Disconnect the power cord before moving or installing the monitor. There is risk of electric shock.
- Installing the monitor on the ceiling or on a slanted wall may result in the monitor falling off, which could lead to injury. Use an authorised LG wall mount and contact the local dealer or qualified personnel.
- To prevent injury, this apparatus must be securely attached to the wall in accordance with the installation instructions.
- Applying excessive force when tightening screws may damage the monitor. Such damage is not covered by the product warranty.
- Use the wall mounting bracket and screws that confirm to VESA standards. Damage caused by the use or misuse of inappropriate components is not covered by the product warranty.
- When measured from the back of the monitor, the length of each installed screw must be 8 mm or less.

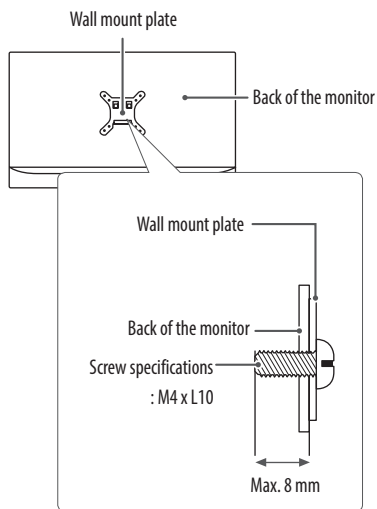

Precautions for Connecting the Power Cord

- Make sure to use the power cord that is provided in the product package. Connect the cord to a grounded power outlet.
- If you need another power cord, please contact your local dealer or the nearest retail store.

USER SETTINGS

[Hot key Settings]

- 1 Press (LED) to activate the LED Control Button. When the LED Control Button is activated, press (OSD) to show the complete OSD Menu.
- 2 Among the LED Control Button, press (◀/▶), to move to the left or right or (▽/△), to move to the bottom or top to go to the [Hot key Settings].
- 3 Configure the options following the instructions that appear in the bottom right corner.
- 4 To configure an upper menu or another item, press the LED Control Button (◀) or press (⊙) to move to the settings.
- 5 In order to exit from the OSD Menu, press the LED Control Button (LED) or press (◀).

The applicable setting options are as shown below.

[Hot key Settings]	Explanation	
[Custom Hotkey]	Designate a hotkey for screen image settings. After setting the hotkey, use the hotkey in the LED Control Button to enable the set menu. ([User Preset], [Picture Mode], [Gamma], [Screen Zoom], [Screen Flip], [Clone screen], [Main/Sub Screen Change], [Off])	
	[Hotkey 1]	Select a function to use with [Hotkey 1].
	[Hotkey 2]	Select a function to use with [Hotkey 2].
	[Option]	Select the item to be enabled for each menu when the hotkey for each item is triggered. ([User Preset], [Picture Mode], [Gamma], [Screen Zoom], [Screen Flip])
[Input Hotkey]	Select an item for input switching. After setting, use the Input hotkey in the LED Control button to switch to the configured input. ([SDI], [DVI-D], [HDMI], [DisplayPort])	

[Hot key Settings]	Explanation		
[PIP/PBP Hotkey]	Select an item for PIP/PBP switching. After setting, use the PIP/PBP hotkey in the LED Control button to switch to the configured PIP/PBP mode. ([PIP 1], [PIP 2], [PBP 1], [PBP 2])		
	[PIP 1]	[PIP 1]	Enable PIP 1 switching.
		[PIP Position]	Set the position for PIP 1 mode.
		[PIP Size]	Set the size for PIP 1 mode.
	[PIP 2]	[PIP 2]	Enable PIP 2 switching.
		[PIP Position]	Set the position for PIP 2 mode.
		[PIP Size]	Set the size for PIP 2 mode.
	[PBP 1]	[PBP 1]	Enable PBP 1 switching.
		[PBP Position]	Set the position for PBP 1 mode.
	[PBP 2]	[PBP 2]	Enable PBP 2 switching.
		[PBP Position]	Set the position for PBP 2 mode.

[Input]

- 1 Press (PIP) to activate the LED Control Button. When the LED Control Button is activated, press (PIP) to show the complete OSD Menu.
- 2 Among the LED Control Button, press (◀/▶) to move to the left or right or (▽/△) to move to the bottom or top to go to the [Input].
- 3 Configure the options following the instructions that appear in the bottom right corner.
- 4 To configure an upper menu or another item, press the LED Control Button (◀) or press (⊙) to move to the settings.
- 5 In order to exit from the OSD Menu, press the LED Control Button (PIP) or press (◀).

The applicable setting options are as shown below.

[Input]	Explanation					
[Input List]	Selects the input mode.					
	PIP / PBP Connection		Sub			
			SDI	DVI-D	HDMI	DisplayPort
	Main	SDI	-	0	0	0
		DVI-D	0	-	0	0
		HDMI	0	0	-	0
		DisplayPort	0	0	0	-
[Input Name]	Set names for each input. ([SDI], [DVI-D], [HDMI], [DisplayPort], [Endoscope], [Laparoscope], [Ultrasound], [C-arm], [PACS], [Surgical Camera], [Surgical Microscope], [Patient Monitor], [Recorder], [Room Camera], [Custom])					
 NOTE When [Custom] is selected, the input name can be configured using the on-screen keyboard.						

[Input]	Explanation	
[Aspect Ratio]	Adjusts the aspect ratio of the screen. ([Full Wide], [Original], [Just Scan]) ! NOTE • The display may look the same for [Full Wide], [Original] and [Just Scan] options at the 3840 x 2160 resolution.	
[Screen Flip]	Adjusts the screen image rotation. ([Rotation], [Mirror], [Off]) ! NOTE • The function is deactivated at the 3PBP / 4PBP Mode.	
[PIP / PBP]	Adjusts display mode for 2 or more screens.	
[PIP Size]	Adjusts the size of PIP. ([Small], [Medium], [Large]) ! NOTE • The screen is zoomed on [Small] Mode.	
[Main/Sub Screen Change]	Converts between the Main and the Sub in the [PIP / PBP] mode.	
[Failover Input Switch]	[Main Input] is changed to [Failover Input] when there is no signal at the [Main Input].	
	[Failover Input Switch]	The [Failover Input Switch] input will be operated or turned off.
	[Main Input]	Select [Main Input] source.
	[Failover Input]	Select [Failover Input] source.

[Input]	Explanation	
[Clone screen]	Turns on or off the [Clone screen] output.	
	[Resolution]	Selects the [Resolution]. • The resolution of the output signal can be set to [Auto], [UHD], [QHD], [FHD], or [SD] depending on the connected device. • When [Auto] is selected, it outputs the maximum resolution based on the input resolution of the connected device.
	[Frame Rate]	Selects the [Frame Rate]. • The refresh rate of the output signal can be set to [Auto], [50Hz], or [60Hz]. • When [Auto] is selected, it outputs the maximum refresh rate based on the input refresh rate of the connected device.
	[Color Gamut]	Selects the [Color Gamut]. • The color gamut of the output signal can be set to [Auto], [REC709], or [REC2020]. • The color gamut can be changed when the picture mode of the monitor is set to [REC709] or [REC2020]. • When [Auto] is selected, the current color gamut displayed on the monitor will be output as it is.

! NOTE

- If not using the [PIP / PBP] feature, [PIP Size], [Main/Sub Screen Change] are deactivated.

[Picture]

- 1 Press (⏏) to activate the LED Control Button. When the LED Control Button is activated, press (⏏) to show the complete OSD Menu.
- 2 Among the LED Control Button, press (◀/▶) to move to the left or right or (▽/△) to move to the bottom or top to go to the [Picture].
- 3 Configure the options following the instructions that appear in the bottom right corner.
- 4 To configure an upper menu or another item, press the LED Control Button (◀) or press (⊙) to move to the settings.
- 5 In order to exit from the OSD Menu, press the LED Control Button (⏏) or press (◀).

The applicable setting options are as shown below.

! NOTE

- The [Picture Mode] that can be set depends on the Input signal.

[Picture Mode] on SDR (Non HDR) signal

[Picture]	Explanation	
[Picture Mode]	[Custom]	Allows the user to adjust each element. The color mode of the main menu can be adjusted.
	[HDR Effect]	Optimizes the screen for high dynamic range.
	[Mono]	Mono (black and white) color mode.
	[REC709]	Optimizes the screen for REC709.
	[REC2020]	Optimizes the screen for REC2020.
	[DICOM]	A mode that optimizes the screen settings so you can view images for medical use.
	[Calibration 1]	Adjusts to the calibrated screen.
	[Calibration 2]	
! NOTE • [Calibration 1]/[Calibration 2]: Please install LG Calibration Studio to run calibration.		

[Picture Mode] on HDR signal

[Picture]	Explanation	
[Picture Mode]	[Custom]	Allows the user to adjust each element.
	 NOTE • HDR content may not display correctly depending on Windows 10 OS settings, please check HDR On/Off settings in Windows. • When the HDR function is turned on, the characters or picture quality may be poor according to Graphic card performance. • While the HDR function is on, screen flickering or cracking may occur when changing the monitor input or power on/off according to Graphic card performance.	

[Picture]	Explanation	
[Picture Adjust]	[Brightness]	Adjusts screen brightness. NOTE - Press the  button to toggle between [Turn on 'Brightness Stabilization']/[Turn off 'Brightness Stabilization']. - When [Brightness Stabilization] is [On], the [Brightness] adjustment function is deactivated. - When [Brightness Stabilization] is [On], [SMART ENERGY SAVING] and [DFC] functions are deactivated. - When [Picture Mode] is set as [DICOM], or [Gamma] is set as [DICOM Gamma Curve] Setting, the [Brightness] adjustment function is deactivated.
		[Contrast] Adjusts the color contrast of the screen.
		[Sharpness] Adjusts the sharpness of the screen.
		[Brightness Stabilization] Maintains the brightness configured of the screen.
	[On]	Automatically adjusts brightness.
	[Off]	Deactivates the applicable function and the user can adjust the brightness.
	[SUPER RESOLUTION+]	[High] Select this option for crystal clear images.
		[Middle] The optimised picture quality is displayed when a user wants images between low and high modes for comfortable viewing.
		[Low] The optimised picture quality is displayed when a user wants smooth and natural images.
		[Off] Select this option for the normal user experience. Deactivates the [SUPER RESOLUTION+] function.
	[Black Level]	Sets up the offset level. (HDMI only) Offset: As a reference for a video signal, this is the darkest color the monitor can display.
		[High] Maintains the current screen contrast range.
		[Low] Reduces the black levels and increases the white levels of the current screen contrast range.

[Picture]	Explanation		
[Picture Adjust]	[DFC]	[On]	Automatically adjusts brightness depending on the screen.
		[Off]	Deactivates the [DFC] function.
	[Response Time]	Sets a response time for displayed pictures based on the movement of the picture on the screen. In a normal environment, it is recommended that you set [Fast]. For images with a lot of motion, it is recommended that you set [Faster]. But, when [Faster] is set, it may cause an afterimage.	
		[Faster]	Sets the response time to Faster.
		[Fast]	Sets the response time to Fast.
		[Normal]	Sets the response time to Normal.
		[Off]	Does not use the response time improvement feature.
	[Black Stabilizer]	Adjusts the black level to clearly see objects on a dark screen.	
		Increasing the [Black Stabilizer] value brightens the low grey levels on the screen. (You can easily distinguish objects on a dark game screen.) Reducing the [Black Stabilizer] value darkens the low grey levels and increases the dynamic contrast on the screen.	
	[Uniformity]	Automatically adjusts the brightness uniformity of the screen. ! NOTE • If [Uniformity] is activated, the screen may become darker.	
		[On]	Activates the [Uniformity] function.
		[Off]	Deactivates the [Uniformity] function.

[Picture]	Explanation	
[Color Adjust]	[Gamma]	Set your own gamma value. ([Gamma 1.8], [Gamma 2.0], [Gamma 2.2], [Gamma 2.4], [Gamma 2.6], [DICOM Gamma Curve]) Higher gamma settings mean a darker image is displayed or vice versa.
	[Color Temp]	Set your own color temperature. ([Custom], [6500K], [7500K], [9300K], [Manual])
		[Custom] Users can customise the red, green and blue colors.
		[6500K] Indicates the screen color with a 6500K red color temperature.
		[7500K] Sets the screen color between red and blue with a 7500K color temperature.
		[9300K] Indicates the screen color with 9300K blue color temperature.
		[Manual] Adjusts the color temperature in 500K increments. (However, supports 9300K instead of 9500K)
	[Red]	You can customise the picture color using red, green and blue colors.
	[Green]	
	[Blue]	
	[Six Color]	Meets the user requirements for colors by adjusting the hue and saturation of the six colors (red, green, blue, cyan, magenta and yellow) and then saving the settings.
		Hue Adjusts the tint of the screen colors.
		Saturation The lower the value, the less saturated. The higher the value, the more saturated.
[Picture Setting Deploy]	Apply the video settings of the current input to the selected input.	
	[SDI]	Enable the application of the video settings for the selected input.
	[DVI-D]	
	[HDMI]	
	[DisplayPort]	
	[Deploy]	The video settings of the current input are applied to the selected input.
[Picture Reset]	Returns color to the default settings.	

[General]

- 1 Press (⏻) to activate the LED Control Button. When the LED Control Button is activated, press (⏻) to show the complete OSD Menu.
- 2 Among the LED Control Button, press (◀/▶) to move to the left or right or (▽/△) to move to the bottom or top to go to the [General].
- 3 Configure the options following the instructions that appear in the bottom right corner.
- 4 To configure an upper menu or another item, press the LED Control Button (◀) or press (⊙) to move to the settings.
- 5 In order to exit from the OSD Menu, press the LED Control Button (⏻) or press (◀).

The applicable setting options are as shown below.

[General]	Explanation	
[Language]	Sets the menu screen to the desired language.	
[SMART ENERGY SAVING]	Conserve energy by using luminance compensation algorithm.	
	[High]	Saves energy using the high-efficiency [SMART ENERGY SAVING] feature.
	[Low]	Saves energy using the low-efficiency [SMART ENERGY SAVING] feature.
	[Off]	Disables the [SMART ENERGY SAVING] function.
[LED Control Button]	Adjusts the ON time of the Control Button. ([Always On], [20Sec Time Out], [10Sec Time Out], [5Sec Time Out])	
[DVI Power Supply]	Supplies power to a device used by connecting to a DVI input terminal in the form of dongle without power. (Max. 0.5 A)	
	[On]	Activates the [DVI Power Supply] function.
	[Off]	Deactivates the [DVI Power Supply] function.

[General]	Explanation
[User Preset]	User Preset allows the user to save or load up to 10 picture quality settings for multiple connected devices in each preset. NOTE You can use User Preset to import or save items in [Picture Adjust] and [Color Adjust] of the [Picture] menu.
	[User Name] Allows the user to change and register a user name (Preset 01 ~ Preset 03, User 01 ~ User 07) as the user wants. The user can enter a user name to be registered by using the screen keyboard. Preset 01 ~ Preset 03 are factory-set user names as samples and the user can change the names.
	[Load User Settings] Allows the user to change the picture quality settings by loading the User Preset settings.
	[Save User Settings] Saves the current picture quality settings in the corresponding User Preset. Preset 01 ~ Preset 03 are factory-set values as samples and the user can change the values. - Preset 01: Use this preset for bluish color. - Preset 02: Use this preset for greenish color and brighter low-gradation. - Preset 03: Use this preset to soften the red tone. - User 01 ~ User 07: Initial values are the same as the factory settings.
	[Default User Settings] Loads the initial basic picture settings.
	[User Preset Reset] Initializes User Preset settings. Initializes the existing user name and user settings to be restored to factory settings (Preset 01 ~ Preset 03, User 01 ~ User 07).
[Auto Screen Off]	Automatically turns off the screen when there is no monitor signal for a set time period.
	[On] Activates [Auto Screen Off] function.
	[Off] Deactivates [Auto Screen Off] function.
[HW Calibration]	[RS-232C] and the [HW Calibration] cannot be used at the same time.
	[On] The HW Calibration function will operate.
	[Off] The HW Calibration function will be turned off.

[General]	Explanation	
[RS-232C]	[RS-232C] and the [HW Calibration] cannot be used at the same time.	
	[RS-232C]	The [RS-232C] function will be operated or turned off.
	[Set ID]	Adjusts [Set ID]. (The scope of adjustment: 1-10)
[Input Compatibility Version]	Please set the [Input Compatibility Version] to match the version of the connected external device.	
	HDMI	[2.0]/[1.4] To enable HDR, set [Input Compatibility Version] to [2.0].
	DisplayPort	[1.4]/[1.2] To enable HDR, set [Input Compatibility Version] to [1.4].
[OSD Lock]	A function for restricting menu configuration and adjustment.	
	[On]	Turns on [OSD Lock].
	[Off]	Turns off [OSD Lock].
	NOTE - All features are disabled except: [Brightness] / [Contrast] / [Brightness Stabilization] in [Picture Adjust] [Input List] / [Input Name] / [Aspect Ratio] / [PIP / PBP] / [Screen Flip] / [Failover Input Switch] / [Clone Screen] in [Input] [OSD Lock] / [Information] / [Open Source Software License] in [General]	
[Information]	The following display information will be shown; [Total Power On Time], [Resolution].	
[Open Source Software License]	You can check the Open Source Software notice information.	
[Reset to Initial Settings]	[Do you want to reset your settings?]	
	[Yes]	Restores the screen settings to the default settings when your monitor was first purchased.
	[No]	Cancels the selection.

TROUBLESHOOTING

Nothing is displayed on the screen.

Problem	Action
Is the monitor's power cord plugged in?	Check if the power cord is correctly plugged into the power outlet.
Is the power indicator on?	Check the power cable connection and turn on the power switch.
Is the power indicator displaying as green?	Check if the input setting is correct. (Input) > [Input]
Is the [Out of Range] message being displayed?	This occurs when signals transferred from the PC (graphics card) are out of the horizontal or vertical frequency range of the monitor. Please see the <PRODUCT SPECIFICATIONS> section of this manual to set the appropriate frequency.
Is the [No Signal] message displayed?	This is displayed when the signal cable between the PC and the monitor is missing or disconnected. Check the cable and reconnect it.

The screen retains an image.

Problem	Action
Does image sticking occur even when the monitor is turned off?	- Displaying a still image for a prolonged time may cause damage to the screen, resulting in the retention of the image. - To extend the lifetime of the monitor, use a screensaver.

The screen is unstable and shakes. / There are shadowy traces left on the screen.

Problem	Action
Did you select the appropriate resolution?	If the selected resolution is HDMI 1080i 60/50 Hz (interlaced), the screen may be flickering. Change the resolution to 1080p or the recommended resolution.

The display color is abnormal.

Problem	Action
Does the display appear discolored (16 colors)?	Set the number of colors to 24 bit (true colors) or higher: In Windows Control Panel > Display > Settings > Color Quality (May differ depending on your operating system.)
Does the display color appear unstable or monochrome?	Check if the signal cable is connected properly. Reconnect the cable or reinsert the PC's graphics card.
Are there spots on the screen?	When using the monitor, pixilated spots (red, green, blue, white or black) may appear on the screen. This is normal for an LCD screen. It is not an error, nor is it related to the monitor's performance.

The CLONE SCREEN output feature is not functioning.

Problem	Action
Is the screen on the monitor connected with Clone Screen not working?	- Check that the connected monitor is turned on. - Check the status of the cable connection.
The image quality of the monitor connected with Clone Screen is poor.	Try adjusting the Clone Screen output settings. (⏏ > [Input] > [Clone screen] to modify the resolution and the refresh rate.)

PRODUCT SPECIFICATIONS

Specifications are subject to change without notice.

The symbol ~ means alternating current, and the symbol — means direct current.

LCD Screen	Type	TFT (Thin Film Transistor) LCD (Liquid Crystal Display) Screen
	Pixel Pitch	0.18159 mm x 0.18159 mm
Resolution	Max Resolution	HDMI / DisplayPort: 3840 x 2160 @ 60 Hz SDI / DVI-D: 1920 x 1080 @ 60 Hz
	Recommended Resolution	HDMI / DisplayPort: 3840 x 2160 @ 60 Hz SDI / DVI-D: 1920 x 1080 @ 60 Hz
Video Signal	Horizontal Frequency	HDMI/DisplayPort: 30 kHz to 135 kHz DVI-D: 30 kHz to 83 kHz
	Vertical Frequency	56 Hz to 61 Hz
Input Connector	SDI IN, DVI-D IN, HDMI IN, DP IN, USB( 5V — 0.9A), USB(), RS-232C, SERVICE ONLY, DC-IN, Potential Equalization Conductor	
Output Connector	SDI OUT, CLONE SCREEN	
Power Sources	Power Rating	19 - 24 V — 6.32 - 5.0 A
	Power Consumption	Max. 100 W* Off Mode: ≤ 0.3 W
AC-DC Adaptor	Manufacturer: Asian Power Devices Inc. (APD) Model: DA-120D19	
	Input: 100-240 V~ 50-60 Hz, 1.8 - 0.7 A Output: 19 V — 6.32 A	
	Classification by protection type against Electric Shock: Class I equipment	
	 NOTE - Connect an AC/DC adapter. - The output of the enclosed adaptor is 19 V. - The product is designed to be used with adaptors that have an output ranging between 19-24 V. Please use the adaptors of medical standards.	


Environmental Conditions	Operating Conditions	Temperature	0 °C to 40 °C
		Humidity	0 % to 80 %
		Pressure	700 hPa to 1060 hPa
	Storage / Transport Conditions	Temperature	-20 °C to 60 °C
		Humidity	0 % to 85 %
		Pressure	500 hPa to 1060 hPa

Monitor Size	
Dimensions (Width x Height x Depth)	751.4 x 469.5 x 65.2 (mm)
Weight (Without Packaging)	9.2 (kg)

Medical Specifications	Classification by protection type against Electric Shock	Class I equipment
	Classification according to the degree of protection against ingress of water or particulate matter	Front: IP45 Except for Front: IP32
	Mode of operation	Continuous Operation
	Environment of Use	This equipment is not suitable for use in the presence of flammable anesthetic or oxygen.

* The Operating Mode of Power Consumption is measured with LGE test standard. (Full White pattern, Maximum resolution)

* The Power Consumption level can be different by operating condition and monitor setting.

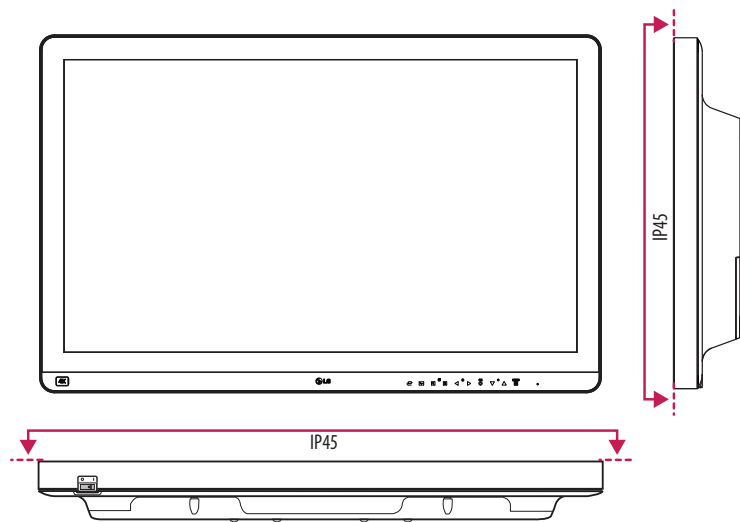

Support Mode (Preset Mode - PC)

DisplayPort

Resolution	Horizontal Frequency (kHz)	Vertical Frequency (Hz)	Polarity (H/V)	Remarks
640 x 480	31.469	59.94	-/-	
800 x 600	37.879	60.317	+/+	
1024 x 768	48.363	60	-/-	
1152 x 864	54.347	60.05	+/+	
1280 x 720	45.00	60.00	+/+	
1280 x 1024	63.981	60.02	+/+	
1600 x 900	60.00	60.00	+/+	
1920 x 1080	67.5	60	+/-	
2560 x 1440	88.79	59.95	+/-	
3840 x 2160	66.66	30	+/-	
3840 x 2160	133.31	60	+/-	for [Input Compatibility Version]: [1.4]/[1.2] Recommended

HDMI

Resolution	Horizontal Frequency (kHz)	Vertical Frequency (Hz)	Polarity (H/V)	Remarks
640 x 480	31.469	59.94	-/-	
800 x 600	37.879	60.317	+/+	
1024 x 768	48.363	60	-/-	
1152 x 864	54.347	60.05	+/+	
1280 x 720	45.00	60.00	+/+	
1280 x 1024	63.981	60.02	+/+	
1600 x 900	60.00	60.00	+/+	
1920 x 1080	67.5	60	+/-	
2560 x 1440	88.79	59.95	+/-	
3840 x 2160	67.5	30	+/-	for [Input Compatibility Version]: [1.4] Recommended
3840 x 2160	135	60	+/-	for [Input Compatibility Version]: [2.0] Recommended

DVI-D

Resolution	Horizontal Frequency (kHz)	Vertical Frequency (Hz)	Polarity (H/V)	Remarks
640 x 480	31.469	59.94	-/-	
800 x 600	37.879	60.317	+/+	
1024 x 768	48.363	60	-/-	
1152 x 864	54.347	60.05	+/+	
1280 x 720	45.00	60.00	+/+	
1280 x 1024	63.981	60.02	+/+	
1600 x 900	60.00	60.00	+/+	
1920 x 1080	67.5	60	+/-	Recommended

Support Mode (Video)

Vertical dimensions	Vertical Frequency (Hz)	DVI-D	HDMI 1.4	HDMI 2.0	DisplayPort	SDI
480i	59.94/60	-	-	-	-	0
480p	59.94/60	0	0	0	-	-
576p	50	0	0	0	-	-
576i	50	-	-	-	-	0
720p	59.94/60	0	0	0	0	0
720p	50	0	0	0	0	0
1080i	59.94/60	0	0	0	-	0
1080p	59.94/60	0	0	0	0	0
1080i	50	0	0	0	-	0
1080p	50	0	0	0	0	0
1080p	23.97/24	-	-	0	-	-
1080p	29.97/30	-	0	0	0	0
2160p	23.98/24	-	-	0	-	-
2160p	25	-	-	0	-	-
2160p	29.97/30	-	-	0	-	-
2160p	50	-	-	0	0	-
2160p	59.94/60	-	-	0	0	-

Power indicator

Mode	LED Color
On Mode	Green

NOTE

- Vertical Frequency: In order to display an image, the screen must be refreshed dozens of times per second, like a fluorescent lamp. The number of times the screen is refreshed per second is called vertical frequency, or refresh rate, and is represented by Hz.
- Horizontal Frequency: The time it takes to display one horizontal line is called the horizontal cycle. If 1 is divided by the horizontal interval, the result is the number of horizontal lines displayed per second. This is called horizontal frequency and is represented by kHz.
- Check if the graphics card's resolution or frequency is within the range allowed by the monitor and in Windows set it to the recommended (optimal) resolution in Control Panel > Display > Settings. (May differ depending on your operating system (OS).)
- Not setting the video card to the recommended (optimal) resolution may result in blurred text, a dimmed screen, a truncated display area or misalignment of the display.
- The setting methods may be different depending on the computer or operating system, and some resolutions may not be available depending on the performance of the graphics card. If this is the case, contact the manufacturer of the computer or graphics card for assistance.
- Normal graphics cards do not support 3840 x 2160 resolution. If the resolution cannot be displayed, contact the manufacturer of your graphics card.

EXTERNAL CONTROLLER SETUP

The actual product may differ from the picture shown.

Connect the RS-232C (serial connector) of the PC to the RS-232C IN connector at the back of the monitor.

Purchase a cable to connect the RS-232C connectors, as the cable is not provided as an accessory.

Use an RS-232C cable to remotely control the monitor (see Figure 1).


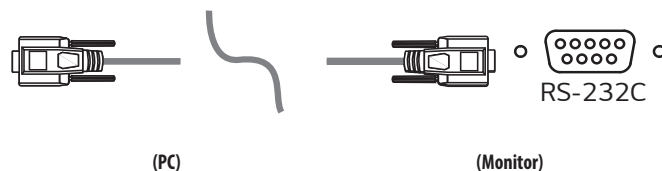


Figure 1: RS-232C Connection Diagram

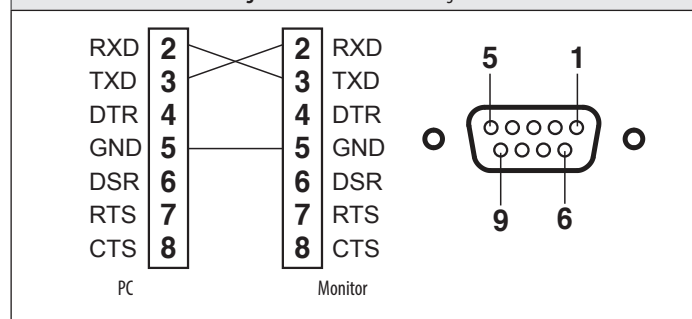

* There are no connections to Pin 1 and Pin 9.

Set ID Function

This function allows you to assign a unique ID to the monitor to control it remotely from your PC. Please refer to the "Transmission/Reception Protocol".

- 1 Press the (⏏) button and select [General] > [RS-232C].
- 2 Set [RS-232C] to [On].
- 3 Use the ◀/▶, ▽/△ buttons to select [Set ID]. Then press the (⊙) button.
- 4 Select a [Set ID] to assign. Then press the (⊙) button. The [Set ID] can be a value from 1 to 10.
- 5 In order to exit from the OSD Menu, press the LED Control Button (⏏) or press (◀).

Communication Parameters

- Baud rate: 9600 bps (UART)
- Data length: 8 bits
- Parity bit: None
- Stop bit: 1 bit
- Communication code: ASCII code
- Crossed (reverse) cable used.

Command Reference List

Command	Command1	Command2	Data (Hexadecimal)
01. Power	k	a	00 ~ 01
02. Screen Mute	k	d	00 ~ 01
03. [Input List] (Main)	x	b	00 ~ FF
[Input List] (Sub)	x	c	00 ~ FF
[Input List] (Sub 2)	x	d	00 ~ FF
[Input List] (Sub 3)	x	e	00 ~ FF
04. [Aspect Ratio] (Main)	x	f	00 ~ 02
[Aspect Ratio] (Sub)	x	g	00 ~ 01
[Aspect Ratio] (Sub 2)	x	h	00 ~ 01
[Aspect Ratio] (Sub 3)	x	i	00 ~ 01
05. [PIP / PBP]	k	n	00 ~ 12
06. [PIP Size]	k	p	00 ~ 02
07. [Main/Sub Screen Change]	m	a	01
08. [Picture Mode]	d	x	00 ~ 14
09. [Brightness]	k	h	00 ~ 64
10. [Contrast]	k	g	00 ~ 64
11. [Sharpness]	k	k	00 ~ 64
12. [Brightness Stabilization]	m	b	00 ~ 01
13. [SUPER RESOLUTION+]	m	c	00 ~ 03
14. [Black Level]	m	d	00 ~ 01
15. [Input Compatibility Version] (HDMI)	m	e	00 ~ 01
16. [Input Compatibility Version] (DP)	m	o	01 ~ 02
17. [DFC]	m	f	00 ~ 01
18. [Response Time]	m	g	00 ~ 03
19. [Black Stabilizer]	m	h	00 ~ 64
20. [Uniformity]	m	i	00 ~ 01

Command	Command1	Command2	Data (Hexadecimal)
21. [Gamma]	m	j	04 ~ 09
22. [Color Temp]	k	u	00 ~ 04
23. [Red]	j	w	00 ~ 64
24. [Green]	j	y	00 ~ 64
25. [Blue]	j	z	00 ~ 64
26. [Language]	f	i	00 ~ 10
27. [SMART ENERGY SAVING]	m	k	00 ~ 02
28. [LED Control Button]	m	l	00 ~ 03
29. [DVI Power Supply]	m	m	00 ~ 01
30. [Auto Screen Off]	m	n	00 ~ 01
31. [OSD Lock]	k	m	00 ~ 01
32. Reset	f	k	00 ~ 02
33. [Screen Flip] (Main)	t	h	00 ~ 02
[Screen Flip] (Sub)	t	i	00 ~ 02
34. [Failover Input Switch]	k	z	00 ~ 01
35. [Main Input]	k	v	00 ~ 03
36. [Failover Input]	k	w	00 ~ 03
37. [Clone screen] (On/Off)	m	q	00 ~ 01
[Clone screen] (Resolution)	m	r	00 ~ 04
[Clone screen] (Frame Rate)	m	s	00 ~ 02
[Clone screen] (Color Gamut)	m	t	00 ~ 02
38. [Picture Setting Deploy]	k	y	00 ~ 0A
39. [Input Name]	m	u	00 ~ 0B
40. [Hot key Settings]	m	v [Hotkey 1]	00 ~ 07
		w [Hotkey 2]	

Transmission/Reception Protocol

Transmission

(Command1)(Command2)(Set ID)(Data)(Cr)

(Command 1): j, k, m, x, t, f, d

(Command 2): This command is used to control the monitor.

(Set ID): Used to identify the monitor being controlled. (Set ID) can be assigned to each monitor under [General] in the Settings Menu.

A value from 1 to 10 can be assigned. By selecting '0' for the (Set ID) value in the protocol format, you can control all connected monitors.

*The value is displayed as base 10 on the OSD menu and used as base 16 (0x00 - 0x63) in the transmission/reception protocol for remote control.

(Data): Transmits a setting value (Data) required for the command described previously. (base 16)

When the Data 'FF' is sent, the setting value corresponding to the specific command is read (Data read mode).

(Cr): Carriage return, which is '0x0D' in ASCII code.

(): Space, which is '0x20' in ASCII code.

OK Acknowledgment

(Command2)()(Set ID)()(OK)(Data)(x)

When the Data has been successfully received, the monitor sends an ACK response signal in the above format. The Data showing the current state is received in the Data read mode. The Data from the PC is simply returned in the Data write mode.

Error Acknowledgment

(Command2)()(Set ID)()(NG)(Data)(x)

When the set receives an abnormal piece of Data for an unsupported function or there is a communication error, it returns ACK in the format above.

Data 00: Illegal code

Actual Data Structure (Base 16 → Base 10)

- See the table below when inserting a base-16 value in (Data).
- * Commands may work differently depending on the model and signal.

01. Power (Command: k a)

- Controls the On/Off of the power supply to the monitor.

Transmission (k)(a)()(Set ID)()(Data)(Cr)

Data

00: Power Off

01: Power On

Ack (a)()(Set ID)()(OK/NG)(Data)(x)

02. Screen Mute (Command: k d)

- Controls the On/Off of the screen of the monitor.

Transmission (k)(d)()(Set ID)()(Data)(Cr)

Data

00: Power Off

01: Power On

Ack (d)()(Set ID)()(OK/NG)(Data)(x)

03. [Input List] (Main) (Command: x b)

- Controls the input mode of the Main.

Transmission (x)(b)()(Set ID)()(Data)(Cr)

Data

D0: SDI

90: HDMI

80: DVI-D

C0: DisplayPort

Ack (b)()(Set ID)()(OK/NG)(Data)(x)

[Input List] (Sub) (Command: x c)

- Controls the input mode of the Sub.

Transmission (x)(c)()(Set ID)()(Data)(Cr)

Data

D0: SDI

90: HDMI

80: DVI-D

C0: DisplayPort

Ack (c)()(Set ID)()(OK/NG)(Data)(x)

[Input List] (Sub 2) (Command: x d)

- Controls the input mode of the Sub 2.

Transmission (x)(d)()(Set ID)()(Data)(Cr)

Data

D0: SDI

90: HDMI

80: DVI-D

C0: DisplayPort

Ack (d)()(Set ID)()(OK/NG)(Data)(x)

[Input List] (Sub 3) (Command: x e)

- Controls the input mode of the Sub 3.

Transmission (x)(e)()(Set ID)()(Data)(Cr)

Data

D0: SDI

90: HDMI

80: DVI-D

C0: DisplayPort

Ack (e)()(Set ID)()(OK/NG)(Data)(x)

04. [Aspect Ratio] (Main) (Command: x f)

- Adjusts the Aspect Ratio of the Main.

Transmission (x)(f)()(Set ID)()(Data)(Cr)

Data

00: [Full Wide]

01: [Original]

02: [Just Scan]

Ack (f)()(Set ID)()(OK/NG)(Data)(x)

[Aspect Ratio] (Sub) (Command: x g)

- Adjusts the Aspect Ratio of the Sub.

Transmission (x)(g)()(Set ID)()(Data)(Cr)

Data

00: [Full Wide]

01: [Original]

Ack (g)()(Set ID)()(OK/NG)(Data)(x)

[Aspect Ratio] (Sub 2) (Command: x h)

- Adjusts the Aspect Ratio of the Sub 2.

Transmission (x)(h)()(Set ID)()(Data)(Cr)

Data

00: [Full Wide]

01: [Original]

Ack (h)()(Set ID)()(OK/NG)(Data)(x)

[Aspect Ratio] (Sub 3) (Command: x i)

- Adjusts the Aspect Ratio of the Sub 3.

Transmission (x)(i)()(Set ID)()(Data)(Cr)

Data

00: [Full Wide]

01: [Original]

Ack (i)()(Set ID)()(OK/NG)(Data)(x)

05. [PIP / PBP] (Command: k n)

- Controls the PIP / PBP mode.

Transmission (k)(n)()(Set ID)()(Data)(Cr)

Data

00: Off

01: 2PBP

05: PIP_LT

06: PIP_RT

07: PIP_LB

08: PIP_RB

0C: 4PBP_QUAD

0D: 3PBP_L1_R2

0E: 3PBP_L2_R1

0F: 4PBP_L1_R3

10: 4PBP_L3_R1

11: 4PBP_T1_B3

12: 4PBP_T3_B1

Ack (n)()(Set ID)()(OK/NG)(Data)(x)

06. [PIP Size] (Command: k p)

- Adjusts the size of PIP.

Transmission (k)(p)()(Set ID)()(Data)(Cr)

Data

00: [Small]

01: [Medium]

02: [Large]

Ack (p)()(Set ID)()(OK/NG)(Data)(x)

07. [Main/Sub Screen Change] (Command: m a)

- Controls the Swap in the PIP/PBP mode.

Transmission (m)(a)()(Set ID)()(Data)(Cr)

Data

01: [Main/Sub Screen Change]

Ack (a)()(Set ID)()(OK/NG)(Data)(x)

08. [Picture Mode] (Command: d x)

- Controls the Picture Mode.

Transmission (d)(x)() (Set ID)() (Data)(Cr)

Data

00: [Custom]

07: [REC2020]

0D: [REC709]

0F: [HDR Effect]

11: [Mono]

12: [DICOM]

13: [Calibration 1]

14: [Calibration 2]

Ack (x)() (Set ID)() (OK/NG)(Data)(x)

09. [Brightness] (Command: k h)

- Adjusts screen brightness.

Transmission (k)(h)() (Set ID)() (Data)(Cr)

Data

Min: 00 - Max: 64

Ack (h)() (Set ID)() (OK/NG)(Data)(x)

10. [Contrast] (Command: k g)

- Adjusts the color contrast of the screen.

Transmission (k)(g)() (Set ID)() (Data)(Cr)

Data

Min: 00 - Max: 64

Ack (g)() (Set ID)() (OK/NG)(Data)(x)

11. [Sharpness] (Command: k k)

- Adjusts the sharpness of the screen.

Transmission (k)(k)() (Set ID)() (Data)(Cr)

Data

Min: 00 - Max: 64

Ack (k)() (Set ID)() (OK/NG)(Data)(x)

12. [Brightness Stabilization] (Command: m b)

- Controls the Brightness Stabilization function.

Transmission (m)(b)() (Set ID)() (Data)(Cr)

Data

00: [On]

01: [Off]

Ack (b)() (Set ID)() (OK/NG)(Data)(x)

13. [SUPER RESOLUTION+] (Command: m c)

- Controls the SUPER RESOLUTION+ function.

Transmission (m)(c)() (Set ID)() (Data)(Cr)

Data

00: [High]

01: [Middle]

02: [Low]

03: [Off]

Ack (c)() (Set ID)() (OK/NG)(Data)(x)

14. [Black Level] (Command: m d)

- Controls the Offset level. (HDMI only)

Transmission (m)(d)()(Set ID)()(Data)(Cr)

Data

00: [High]

01: [Low]

Ack (d)()(Set ID)()(OK/NG)(Data)(x)

15. [Input Compatibility Version] (HDMI) (Command: m e)

- Controls the [Input Compatibility Version]. (HDMI only)

Transmission (m)(e)()(Set ID)()(Data)(Cr)

Data

00: [2.0]

01: [1.4]

Ack (e)()(Set ID)()(OK/NG)(Data)(x)

16. [Input Compatibility Version] (DP) (Command: m o)

- Controls the [Input Compatibility Version]. (DisplayPort only)

Transmission (m)(o)()(Set ID)()(Data)(Cr)

Data

01: [1.2]

02: [1.4]

Ack (o)()(Set ID)()(OK/NG)(Data)(x)

17. [DFC] (Command: m f)

- Controls the DFC function.

Transmission (m)(f)()(Set ID)()(Data)(Cr)

Data

00: [On]

01: [Off]

Ack (f)()(Set ID)()(OK/NG)(Data)(x)

18. [Response Time] (Command: m g)

- Controls the Response Time.

Transmission (m)(g)()(Set ID)()(Data)(Cr)

Data

00: [Faster]

01: [Fast]

02: [Normal]

03: [Off]

Ack (g)()(Set ID)()(OK/NG)(Data)(x)

19. [Black Stabilizer] (Command: m h)

- Controls the Black Sharpness Optimization function.

Transmission (m)(h)() (Set ID)()(Data)(Cr)

Data

Min: 00 - Max: 64

Ack (h)() (Set ID)()(OK/NG)(Data)(x)

20. [Uniformity] (Command: m i)

- Controls the Uniformity function.

Transmission (m)(i)() (Set ID)()(Data)(Cr)

Data

00: [On]

01: [Off]

Ack (i)() (Set ID)()(OK/NG)(Data)(x)

21. [Gamma] (Command: m j)

- Adjusts the Gamma settings.

Transmission (m)(j)() (Set ID)()(Data)(Cr)

Data

04: [Gamma 1.8]

05: [Gamma 2.0]

06: [Gamma 2.2]

07: [Gamma 2.4]

08: [Gamma 2.6]

09: [DICOM Gamma Curve]

Ack (j)() (Set ID)()(OK/NG)(Data)(x)

22. [Color Temp] (Command: k u)

- Adjusts the color Temp.

Transmission (k)(u)() (Set ID)()(Data)(Cr)

Data

00: [Manual]

01: [Custom]

02: [6500K]

03: [7500K]

04: [9300K]

Ack (u)() (Set ID)()(OK/NG)(Data)(x)

23. [Red] (Command: j w)

- Adjusts the red.

Transmission (j)(w)() (Set ID)()(Data)(Cr)

Data

Min: 00 - Max: 64

Ack (w)() (Set ID)()(OK/NG)(Data)(x)

24. [Green] (Command: j y)

- Adjusts the green.

Transmission (j)(y)() (Set ID)()(Data)(Cr)

Data

Min: 00 - Max: 64

Ack (y)() (Set ID)()(OK/NG)(Data)(x)

25. [Blue] (Command: j z)

- Adjusts the blue.

Transmission (j)(z)()(Set ID)()(Data)(Cr)

Data

Min: 00 - Max: 64

Ack (z)()(Set ID)()(OK/NG)(Data)(x)

26. [Language] (Command: f i)

- Adjusts the language of the Menu screen.

Transmission (f)(i)()(Set ID)()(Data)(Cr)

Data

00 - 10: English - Korean (17 languages)

Ack (i)()(Set ID)()(OK/NG)(Data)(x)

27. [SMART ENERGY SAVING] (Command: m k)

- Adjusts the SMART ENERGY SAVING.

Transmission (m)(k)()(Set ID)()(Data)(Cr)

Data

00: [High]

01: [Low]

02: [Off]

Ack (k)()(Set ID)()(OK/NG)(Data)(x)

28. [LED Control Button] (Command: m l)

- Adjusts the LED ON time of the Control Button.

Transmission (m)(l)()(Set ID)()(Data)(Cr)

Data

00: [Always On]

01: [20Sec Time Out]

02: [10Sec Time Out]

03: [5Sec Time Out]

Ack (l)()(Set ID)()(OK/NG)(Data)(x)

29. [DVI Power Supply] (Command: m m)

- Controls the DVI Power Supply function.

Transmission (m)(m)()(Set ID)()(Data)(Cr)

Data

00: [On]

01: [Off]

Ack (m)()(Set ID)()(OK/NG)(Data)(x)

30. [Auto Screen Off] (Command: m n)

- Adjusts the time to automatically turn the screen off when there is no monitor signal for a set period of time.

Transmission (m)(n)()(Set ID)()(Data)(Cr)

Data

00: [On]

01: [Off]

Ack (n)()(Set ID)()(OK/NG)(Data)(x)

31. [OSD Lock] (Command: k m)

- Controls the OSD Lock function.

Transmission (k)(m)()(Set ID)()(Data)(Cr)

Data

00: [Off]

01: [On]

Ack (m)()(Set ID)()(OK/NG)(Data)(x)

32. Reset (Command: f k)

- Controls the Reset operation.

Transmission (f)(k)()(Set ID)()(Data)(Cr)

Data

00: [Picture Reset]

01: Factory Reset

02: [User Preset Reset]

Ack (k)()(Set ID)()(OK/NG)(Data)(x)

33. [Screen Flip] (Main) (Command: t h)

- Adjusts the screen image rotation of Main screen.

Transmission (t)(h)()(Set ID)()(Data)(Cr)

Data

00: [Off]

01: [Mirror]

02: [Rotation]

Ack (h)()(Set ID)()(OK/NG)(Data)(x)

[Screen Flip] (Sub) (Command: t i)

- Adjusts the screen image rotation of Sub screen.

Transmission (t)(i)()(Set ID)()(Data)(Cr)

Data

00: [Off]

01: [Mirror]

02: [Rotation]

Ack (i)()(Set ID)()(OK/NG)(Data)(x)

34. [Failover Input Switch] (Command: k z)

- Controls the Failover Input Switch function.

Transmission (k)(z)()(Set ID)()(Data)(Cr)

Data

00: [Off]

01: [On]

Ack (z)()(Set ID)()(OK/NG)(Data)(x)

35. [Main Input] (Command: k v)

- Controls the input of the Failover Main.

Transmission (k)(v)()(Set ID)()(Data)(Cr)

Data

00: SDI

01: DVI-D

02: HDMI

03: DisplayPort

Ack (v)()(Set ID)()(OK/NG)(Data)(x)

36. [Failover Input] (Command: k w)

- Controls the input of the Failover.

Transmission (k)(w)()(Set ID)()(Data)(Cr)

Data

00: SDI

01: DVI-D

02: HDMI

03: DisplayPort

Ack (w)()(Set ID)()(OK/NG)(Data)(x)

37. [Clone screen] (On/Off) (Command: m q)

- Controls the clone screen feature.

Transmission (m)(q)()(Set ID)()(Data)(Cr)

Data

00: [On]

01: [Off]

Ack (q)()(Set ID)()(OK/NG)(Data)(x)

[Clone screen] (Resolution) (Command: m r)

- Sets the resolution for the clone screen.

Transmission (m)(r)()(Set ID)()(Data)(Cr)

Data

00: [Auto]

01: SD

02: FHD

03: QHD

04: UHD

Ack (r)()(Set ID)()(OK/NG)(Data)(x)

[Clone screen] (Frame Rate) (Command: m s)

- Sets the frame rate for the clone screen.

Transmission (m)(s)()(Set ID)()(Data)(Cr)

Data

00: [Auto]

01: [50Hz]

02: [60Hz]

Ack (s)()(Set ID)()(OK/NG)(Data)(x)

[Clone screen] (Color Gamut) (Command: m t)

- Sets the color gamut for the clone screen.

Transmission (m)(t)()(Set ID)()(Data)(Cr)

Data

00: [Auto]

01: [REC709]

02: [REC2020]

Ack (t)()(Set ID)()(OK/NG)(Data)(x)

38. [Picture Setting Deploy] (Command: k y)

- Configure and apply video setting adjustments.

Transmission (k)(y)()(Set ID)()(Data)(Cr)

Data

00: [SDI]

02: [HDMI]

0A: [Deploy]

01: [DVI-D]

03: [DisplayPort]

Ack (y)()(Set ID)()(OK/NG)(Data)(x)

39. [Input Name] (Command: m u)

- Set the name of the current input.

Transmission (m)(u)()(Set ID)()(Data)(Cr)

Data

00: Default settings

02: [Laparoscope]

04: [C-arm]

06: [Surgical Camera]

08: [Patient Monitor]

0A: [Room Camera]

01: [Endoscope]

03: [Ultrasound]

05: [PACS]

07: [Surgical Microscope]

09: [Recorder]

0B: [Custom]

Ack (u)()(Set ID)()(OK/NG)(Data)(x)

40. [Hot key Settings]

Hotkey 1: (Command: m v), Hotkey 2: (Command: m w)

- Set the items for Hotkey Settings 1 and 2.

Hotkey 1

Transmission (m)(v)()(Set ID)()(Data)(Cr)

Hotkey 2

Transmission (m)(w)()(Set ID)()(Data)(Cr)

Data

00: [Off]

02: [Picture Mode]

04: [Screen Zoom]

06: [Clone screen]

01: [User Preset]

03: [Gamma]

05: [Screen Flip]

07: [Main/Sub Screen Change]

Hotkey 1

Ack (v)()(Set ID)()(OK/NG)(Data)(x)

Hotkey 2

Ack (w)()(Set ID)()(OK/NG)(Data)(x)

PREVENTIVE MAINTENANCE PLAN – RECOMMENDATION

Annual:

- Perform Electrical Safety test
- Perform QA task according to local Radiologic Regulation
- Recommended annual QA procedure

Procedure	Interval
Viewing Condition Check and Setting	Annually
Display Calibration	Annually
Image Quality Check - SMPTE Pattern	Annually

Regularly, e.g. once per Month:

- Check the integrity of the power cord and inspect its routing, so that it is not under the risk of being punched or cut.
- Check the integrity of the protective earth connection
- Check and cleaning for contamination of ventilation opening on rear housing.
- To keep the optimal viewing conditions of LCD PANEL, periodically check that it is correctly operation

Procedure	Interval
Cleaning Display	Monthly
Viewing Condition	Monthly
Image Quality Check - SMPTE Pattern	Monthly

Daily:

- To maintain the screen, clean by keeping the display screen free from any objects (dust, finger print, etc.) of the display (Recommended equipment: Lint free cloth.)
- Check housing for physical Damage
- Check cabling for Damage and trip hazard
- Perform visual check to verify Display Performance

CYBER SECURITY MANAGEMENT SERVICE


LGE operates an internal process for various cyber security-related issues that occur in the actual use environment.

The purpose of the process operation is to carry out plans, countermeasures, and response activities to respond to security issues when security issues such as security vulnerabilities in our products and Product integration systems occur.

For initial application, use the PSRC official email account.

- PSRC email: product.security@lge.com

A typical response process is as follows:

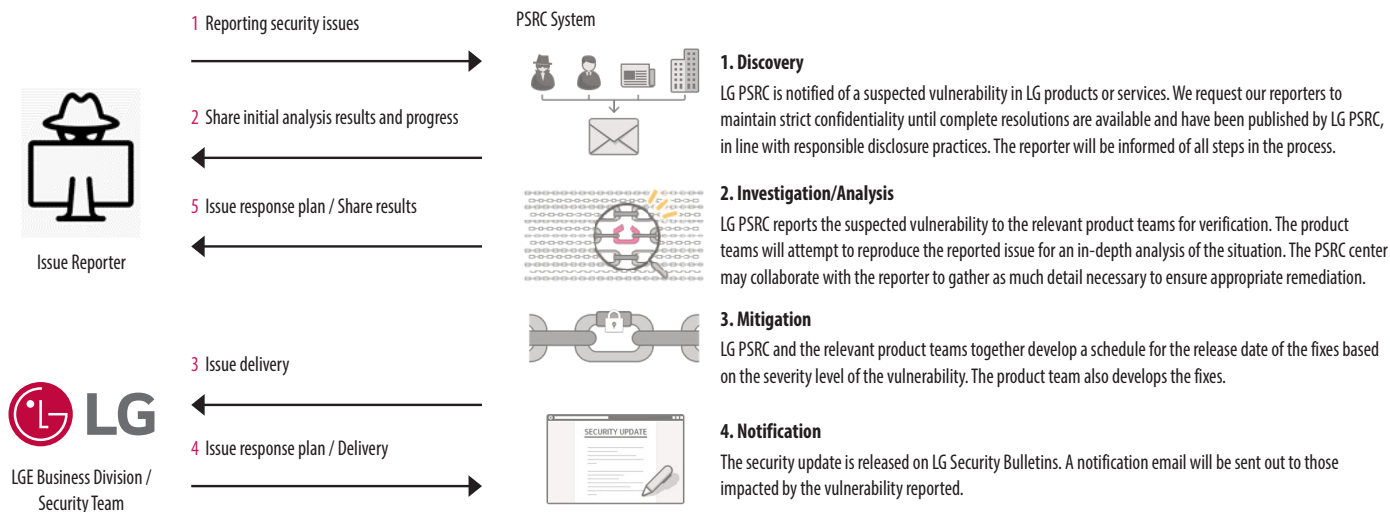

- After the issue has been improved, if it is necessary to update the program, such as releasing a new installation file or applying some patch files, it is officially distributed through the head office sales/SE personnel along with the application guide to the person in charge of the corporation or the issue site.
- After end of support of product life, security patch can be offered for cyber security incidents prevention.



The model and serial number of the product are located on the back and on one side of the product.
Record them below in case you ever need service.

MODEL

SERIAL NO.

(Australia only)

This product must be installed by a professional installer.

After the installation of the product in the customer premises, fully remove all product packaging (including any expanded or moulded plastic packaging materials) and recycle or dispose of such packaging responsibly. Do not dispose any expanded plastic packaging in the household bin.

WARNING: This equipment is compliant with Class A of CISPR 32. In a residential environment this equipment may cause radio interference.