KULANIM KILAVUZU

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---

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The screenshot shows the LG DXD Calibration software interface. The top menu bar has tabs for 'Kalibrasyon', 'User BPM' (highlighted with a red box and number 1), 'Onay', and 'EI'. The main display area shows a black image with a yellow circle in the center. The left panel contains a histogram and a 'Sıfırla' button. The right panel has a 'Geçmiş' (History) section with a dropdown menu showing 'Manual\_Map' and a list of files, with 'BPM\_20191120144048.raw' selected (highlighted with a red box and number 3). Below this is a 'Piksel' (Pixel) section with a grid of pixel values (highlighted with a red box and number 4). The bottom of the interface shows a table with columns for 'Görüntü Yüklendi', 'Min=0, Maks=349, Ortas=165, Ortalama=168,79, Standart=17,65, Bozuk Piksel=72', and 'Detaylar'. The table contains three rows of data for the date 2019-11-20, all at 14:40:49, with the action 'User BPM'.

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The screenshot shows the LG DXD Calibration software interface. The top menu bar includes 'Kalibrasyon', 'User BPM', 'Onay', and 'EI' (highlighted with a red box and number 1). The main window is divided into several sections: 'Navigatör' (Navigator) on the left, 'Histogram' (Histogram) in the middle-left, 'BPM Analizi' (BPM Analysis) at the bottom-left, and a large central area for the image. On the right side, there is a 'Geçmiş' (History) button (highlighted with a red box and number 2), a 'Görüntü' (Image) button (highlighted with a red box and number 3), and a 'Doz' (Dose) column header (highlighted with a red box and number 4). At the bottom right, there is a 'Görüntü' (Image) button (highlighted with a red box and number 5). The bottom section of the interface contains a table with the following data:

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| DXD Monitoring System                                                                                                                                                                                                            |                                                                                                                                                                                                                                                                                                 |                                                                                                                                                                                                                                                                                                          |                                                                    |
|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--------------------------------------------------------------------|
| Product Information                                                                                                                                                                                                              | Network Information                                                                                                                                                                                                                                                                             | Battery                                                                                                                                                                                                                                                                                                  | ETC                                                                |
| <b>Software Version</b><br>Firmware Ver. 3.00.00<br><br><b>Manufacturing Date</b><br>2017. 06. 03<br><br><b>Installation Date</b><br>0000. 00. 00<br><br><b>Model Number</b><br>17HK700G<br><br><b>Serial Number</b><br>12345678 | <b>Status</b><br> Wired Connected<br><br><b>IP</b><br>10.10.10.100<br><br><b>SSID</b><br>N/A<br><br><b>Netmask</b><br>255.255.255.0<br><br><b>Gateway</b><br>10.10.10.1<br><br><b>Mac</b><br>92:B7:1A:7B:FD:95 | <b>Status</b><br> Battery Disconnected, Using Power supply<br> N/A<br><br><b>Auto Sleep</b><br>Off<br><br><b>Auto Power-Off</b><br>Off | <b>Bright Image Count</b><br>0<br><br><b>Dark Image Count</b><br>0 |

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### English

#### Removal of waste batteries and accumulators (Product with embedded battery ONLY)

In case this product contains a battery incorporated within the product which cannot be readily removed by end-users, LG recommends that only qualified professionals remove the battery, either for replacement or for recycling at the end of this product's working life. To prevent damage to the product, and for their own safety, users should not attempt to remove the battery and should contact LG Service Helpline, or other independent service providers for advice.

Removal of the battery will involve dismantling of the product case, disconnection of the electrical cables/contacts, and careful extraction of the battery cell. If you need the instructions for qualified professionals on how to remove the battery safely, please visit <http://www.lge.com/global/sustainability/environment/take-back-recycling>

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### ENGLISH

#### Disposal of your old appliance

- 1 This crossed-out wheeled bin symbol indicates that waste electrical and electronic products (WEEE) should be disposed of separately from the municipal waste stream.
- 2 Old electrical products can contain hazardous substances so correct disposal of your old appliance will help prevent potential negative consequences for the environment and human health. Your old appliance may contain reusable parts that could be used to repair other products, and other valuable materials that can be recycled to conserve limited resources.
- 3 You can take your appliance either to the shop where you purchased the product, or contact your local government waste office for details of your nearest authorised WEEE collection point. For the most up to date information for your country please see [www.lg.com/global/recycling](http://www.lg.com/global/recycling)

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## WEEE

### Disposal of your old appliance

- 1 This crossed-out wheeled bin symbol indicates that waste electrical and electronic products (WEEE) should be disposed of separately from the municipal waste stream.
- 2 Old electrical products can contain hazardous substances so correct disposal of your old appliance will help prevent potential negative consequences for the environment and human health. Your old appliance may contain reusable parts that could be used to repair other products, and other valuable materials that can be recycled to conserve limited resources.
- 3 You can take your appliance to any of the authorized centers for collection, If you want to know an updated list of countries that have this service, visit the page <https://www.lg.com/global/take-back-recycling-global-network-south-america>

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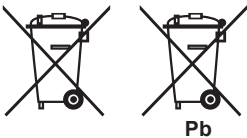
## Environmental information

### Disposal of your old appliance (as per e-waste Rules)

1. When this crossed out wheeled bin symbol is depicted on the product and its operator's manual, it means the product is covered by E-Waste Management Rules India published by MoEF & CC, Govt. of India and are meant to be recycled, dismantled, refurbished or disposed off.
2. Do's :
  - a. The product is required to be handed over only to the authorized recycler for disposal.
  - b. Keep the product in isolated area, after it becomes non-functional/unrepairable so as to prevent its accidental breakage.
- Don'ts :
  - a. The product should not be opened by the user himself/herself, but only by authorized service personnel.
  - b. The product is not meant for re-sale to any unauthorized agencies/scrap dealer/kabariwalah.
  - c. The product is not meant for mixing into household waste stream.
  - d. Do not keep any replaced spare part(s) from the product in exposed area.
3. Any disposal through unauthorized agencies/person will attract action under Environment (Protection) Act 1986.
4. This product is complied with the requirement of Hazardous Substances as specified under E-Waste Management Rules India published by MoEF & CC, Govt. of India.
5. To locate a nearest collection centre or call for pick-up (limited area only) for disposal of this appliance, please contact No. 08069379999, Toll Free No. 1800-315-9999 for details. All collection centre and pick up facilities are done by third parties with LG Electronics India Ltd. Merely as a facilitator. For more detailed information , please visit : <http://www.lg.com/in>.

## Battery

Pb

1. This symbol may be combined with chemical symbols for mercury(Hg), cadmium(Cd) or lead(Pb) if the battery Contains more than 0.0005% of mercury, 0.002% of cadmium or 0.004% of lead.
2. All batteries/accumulators should be disposed separately from the municipal waste stream via designated collection facilities appointed by the government or the local authorities.
3. The correct disposal of Your old batteries/accumulators will help to prevent potential negative consequences for the environment, animal and human health.

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INDIA ONLY

## Packaging Type Details as per PWM Rules

### Brand Owner (LG Electronics India Ltd.)

| Packaging type   | Company name             | Registration number        | Thickness         |
|------------------|--------------------------|----------------------------|-------------------|
| Plastic Sheet    | LG Electronics India Ltd | BO-13-000-07-AAACL1745Q-22 | 0.05 mm and above |
| Multilayered bag | LG Electronics India Ltd |                            | N/A               |

### Importer (LG Electronics India Ltd.)

| Packaging type   | Company name             | Registration number        | Thickness         |
|------------------|--------------------------|----------------------------|-------------------|
| Plastic Sheet    | LG Electronics India Ltd | IM-13-000-07-AAACL1745Q-22 | 0.05 mm and above |
| Multilayered bag | LG Electronics India Ltd |                            | N/A               |

**LG Electronics EU Importer names and addresses / Имена и адреси на вносителите на LG Electronics в ЕС / Nazivi i adrese uvoznika LG Electronics u EU / Nazivi i adrese uvoznika LG Electronics u EU / Názvy a adresy dovozců LG Electronics v EU / Navne og adresser på importører af LG Electronics i EU / Namen und Adressen der LG Electronics EU-Importeure / Ονόματα και διευθύνσεις εισαγωγέων της LG Electronics EU / Nombres y direcciones de importadores para la UE de LG Electronics / LG Electronics EL maaletoojate nimed ja aadressid / LG Electronics – EU:n maahantuojaen nimet ja osoitteet / Noms et adresses des importateurs pour LG Electronics EU / Ainmneacha agus seolaí Iompórtálaí AE LG Electronics / Nazivi i adrese uvoznika proizvoda tvrtke LG Electronics u Europskoj uniji / Az LG Electronics európai uniós importőreinek neve és címe / Nöfn og heimilisföng LG Electronics innflytjenda innan ESB / Nomi e indirizzi degli importatori UE di LG Electronics / LG Electronics ES importuotojų pavadinimai ir adresai / LG Electronics ES importētāju nosaukumi un adreses / Имиња и адреси на увозници на LG Electronics за ЕУ / LG Electronics: ismijiet u indirizzi tal-importaturi tal-UE / Namen en adressen van importeurs voor LG Electronics EU / Navn og adresser for EU-importører for LG Electronics / Nazwy i adresy importerów produktów LG Electronics w Europie / Nomes e endereços dos importadores para LG Electronics EU / Numele și adresele Importatorului UE al LG Electronics / Názvy a adresy dovozcov LG Electronics do EÚ / Imena in naslovi uvoznikov za LG Electronics EU / Emrat dhe adresat e importuesve të LG Electronics në BE / Nazivi i adrese EU uvoznika kompanije LG Electronics / Namn och adresser för LG Electronics EU-importörer**

|                                   |                                                                                                |
|-----------------------------------|------------------------------------------------------------------------------------------------|
| LG Electronics Benelux B.V.       | Krijgsman 1, 1186 DM Amstelveen, The Netherlands                                               |
| LG Electronics Deutschland GmbH   | Alfred-Herrhausen-Allee 3-5, 65760, Eschborn, Germany                                          |
| LG Electronics France, SAS        | Immeuble Workstation, 25 quai du Président Paul Doumer, 92 400 Courbevoie, France              |
| LG Electronics España S.A.U.      | Calle Chile 1, 28290, Las Rozas, Madrid, Spain                                                 |
| LG Electronics Portugal S.A.      | Taguspark, Av. Jacques Delors 3, Edifício Inovacao I, Corpo 1, 2740-122, Porto Salvo, Portugal |
| LG Electronics Hellas S.A.        | Ethnarhou Makariou str. 1, 17501, Delta Paleo Faliro, Greece                                   |
| LG Electronics Italia S.p.A.      | Via Aldo Rossi 4, 20149, Milan, Italy                                                          |
| LG Electronics Magyar Kft.        | Konyves K. krt.3/a, 1097, Budapest, Hungary                                                    |
| LG Electronics Polska Sp. z o.o., | Wołoska 22, 02-675, Warszawa, Poland                                                           |
| LG Electronics Nordic AB          | Esbogatan 18, 164 94, Kista, Sweden                                                            |
| LG Electronics Latvia SIA         | Gustava Zemgale 74A, "Henrihs", Riga, LV-1039, Latvia                                          |

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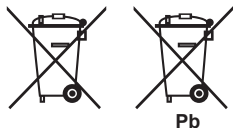
# BATTERY

## ENGLISH

### Disposal of waste batteries/accumulators

This symbol may be combined with chemical symbols for mercury(Hg), cadmium(Cd) or lead(Pb) if the battery Contains more than 0.0005% of mercury, 0.002% of cadmium or 0.004% of lead. All batteries/accumulators should be disposed separately from the municipal waste stream via designated collection facilities appointed by the government or the local authorities.

The correct disposal of Your old batteries/accumulators will help to prevent potential negative consequences for the environment, animal and human health.

For more detailed information about disposal of Your old batteries/accumulators, please contact Your city office, waste disposal service or the shop where You purchased the product. (<http://www.lg.com/global/sustainability/environment/take-back-recycling/global-network-europe>)

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## USA Only (Product with embedded Lithium-ion battery ONLY)

### For USA

In case rechargeable battery is included in this product, the entire device should be recycled in compliance with rechargeable battery recycling standards because of the internal battery. To dispose of properly, call (800) 822-8837 or visit [www.call2recycle.org](http://www.call2recycle.org).

## FOR SAUDI ARABIA (PRODUCT WITH BATTERY ONLY)

- 1 All batteries should be disposed separately from the municipal waste stream via designated collection facilities appointed by the government or the local authorities.
- 2 The correct disposal of your old batteries will help to prevent potential negative consequences for the environment, animal and human health.
- 3 For more detail information about disposal of your old batteries, please contact your city office, waste disposal service or the shop where you purchased the product. Also You can contact our LG Support Representative Call center from below Link  
[https://www.lg.com/sa\\_en/support/contact/telephone](https://www.lg.com/sa_en/support/contact/telephone)

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[https://www.lg.com/sa\\_en/support/contact/telephone](https://www.lg.com/sa_en/support/contact/telephone)

# WIRELESS

## FCC Notice (For USA)

This equipment has been tested and found to comply with the limits for a Class A digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference when the equipment is operated in a commercial environment. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instruction manual, may cause harmful interference to radio communications. Operation of this equipment in a residential area is likely to cause harmful interference in which case the user will be required to correct the interference at his own expense.

Caution: Any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) this device may not cause harmful interference, and
- (2) this device must accept any interference received, including interference that may cause undesired operation.

## FCC Radio Frequency Interference Requirements (for UNII devices)

High power radars are allocated as primary users of the 5.25 to 5.35 GHz and 5.65 to 5.85 GHz bands. These radar stations can cause interference with and/or damage this device. This device cannot be co-located with any other transmitter.

Transmitters in the 5.925-7.125 GHz band are prohibited from operating to control or communicate with unmanned aircraft systems.

## FCC RF Radiation Exposure Statement

### **[For having wireless function (WLAN, Bluetooth,...)]**

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

## CANADA(IC) Notice

### Industry Canada Statement

#### **[For having wireless function (WLAN, Bluetooth,...)]**

This device contains licence-exempt transmitter(s)/receiver(s) that comply with Innovation, Science and Economic Development Canada's licence-exempt RSS(s). Operation is subject to the following two conditions:

- 1) This device may not cause interference.
- 2) This device must accept any interference, including interference that may cause undesired operation of the device.

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## IC Radiation Exposure Statement

### **[For having wireless function (WLAN, Bluetooth,...)]**

This equipment complies with IC radiation exposure limits set forth for an uncontrolled environment.

NOTE: THE MANUFACTURER IS NOT RESPONSIBLE FOR ANY RADIO OR TV INTERFERENCE CAUSED BY UNAUTHORIZED MODIFICATIONS TO THIS EQUIPMENT. SUCH MODIFICATIONS COULD VOID THE USER'S AUTHORITY TO OPERATE THE EQUIPMENT.

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## RSS-247 Requirement

### **[For product having the wireless function using 5 GHz frequency bands]**

- i. the device for operation in the band 5150–5250 MHz is only for indoor use to reduce the potential for harmful interference to co-channel mobile satellite systems;
- ii. for devices with detachable antenna(s), the maximum antenna gain permitted for devices in the bands 5250-5350 MHz and 5470-5725 MHz shall be such that the equipment still complies with the e.i.r.p. limit;
- iii. for devices with detachable antenna(s), the maximum antenna gain permitted for devices in the band 5725-5850 MHz shall be such that the equipment still complies with the e.i.r.p. limits as appropriate; and

iv. [for devices operating in the band 5250-5350 MHz having an e.i.r.p. greater than 200 mW] antenna type(s), antenna models(s), and worst-case tilt angle(s) necessary to remain compliant with the e.i.r.p. elevation mask requirement set forth in section 6.2.2.3 of RSS-247 shall be clearly indicated.

Users should also be advised that high-power radars are allocated as primary users (i.e. priority users) of the bands 5250-5350 MHz and 5650-5850 MHz and that these radars could cause interference and/or damage to LE-LAN devices.

#### **[For product having the wireless function using 6 GHz frequency bands]**

- (1) Devices shall not be used for control of or communications with unmanned aircraft systems.
- (2) Devices shall not be used on oil platforms.
- (3) Devices shall not be used on aircraft, except for the low-power indoor access points, indoor subordinate devices, low-power client devices, and very low-power devices operating in the 5925-6425 MHz band, that may be used on large aircraft as defined in the Canadian Aviation Regulations, while flying above 3,048 meters (10,000 feet).
- (4) Devices shall not be used on automobiles.
- (5) Devices shall not be used on trains.
- (6) Devices shall not be used on maritime vessels.

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## Australia and New Zealand region only

The use of the 5,150 – 5,350 MHz band is restricted to indoor use only.

This restriction exists in the countries Australia (AU) and New Zealand (NZ). (For Wi-Fi/Bluetooth built-in model or Wi-Fi built-in model)

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## Conformity Notice

### English

[For Equipment Under Radio Equipment Directive]

Hereby, LG Electronics declares that the radio equipment type [Flat Panel Digital X-ray Detector] is in compliance with Directive 2014/53/EU. The full text of the EU declaration of conformity is available at the following internet address:

<http://www.lg.com/global/support/cedoc/cedoc#>

LG Electronics Deutschland GmbH

Alfred-Herrhausen-Allee 3-5, 65760 Eschborn, Germany

The use of the 5150 – 5350 MHz band is restricted to indoor use only. This restriction exists in Belgium (BE), Bulgaria (BG), Czech Republic (CZ), Denmark (DK), Germany (DE), Estonia (EE), Ireland (IE), Greece (EL), Spain (ES), France (FR), Croatia (HR), Italy (IT), Cyprus (CY), Latvia (LV), Lithuania (LT), Luxembourg (LU), Hungary (HU), Malta (MT), Netherlands (NL), Austria (AT), Poland (PL), Portugal (PT), Romania (RO), Slovenia (SI), Slovakia (SK), Finland (FI), Sweden (SE), Northern Ireland (UK(NI)), Switzerland (CH), Iceland (IS), Liechtenstein (LI), Türkiye (TR) and Norway (NO). (For Wi-Fi/Bluetooth built-in model or Wi-Fi built-in model)

WARNING: This equipment is compliant with Class A of CISPR 32. In a residential environment this equipment may cause radio interference.

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By using this product, you agree that all disputes between you and LG arising out of or relating in any way to this product (including but not limited to warranty disputes) shall be resolved through binding arbitration on an individual basis, unless the laws of your state or territory do not permit that, or, in other jurisdictions, if you choose to opt out. Additional information regarding the terms of the arbitration agreement (including details on the procedure for resolving disputes) is available at [www.lg.com/us/arbitration](http://www.lg.com/us/arbitration) (USA) and/or your owner's manual or warranty.

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Copyright © Tüm Hakları Saklıdır